MAXIMA Elite Multipurpose Solution

- The instruction contains important product use and safety information. Please read carefully and retain.
- Multipurpose Solution MAXIMA Elite is an all-in-one solution that cleans, removes protein and lipids, rinses, disinfects, stores and lubricates all soft contact lenses, include silicon hydrogel lenses (without further mechanical cleansing, "NO RUB").
- Isotonic aqueous solution contains Remoprog - the active ingredient, moisturising agent Aqua Shield and polyhexamethyleneguanid (PHMB) 0,0001% - the preservative.
- Sterile.
- Manufactured under aseptic technology.
- MAXIMA Elite solution offers everything that you need for correct and complete soft contact lens care without further mechanical cleansing, ("NO RUB").

Application instruction:

- Open the bottle turning the screw-top anti-clockwise to unseal the sealing shutter.
- Always wash your hands and dry them with a clean towel before handling your lenses.
- Open the container and fill it with MAXIMA Elite solution up to the mark.
- Always start lens removal with the same eye to avoid mixing up the lenses.
- Remove the lens and rinse it immediately with MAXIMA Elite solution from both sides carefully.
- Put the lens into the container and close the container with the screw-top bearing the mark corresponding to the lens.
- Repeat the procedure with the other lens.
- Shake the container with lenses for 15 seconds at the least.
- Allow your lenses to soak in the solution for at least 4 hours for disinfection.
- After disinfection rinse both sides of the lens with MAXIMA Elite solution.
- Now your lenses are clean and disinfected and may be inserted directly into your eyes.

- Pour the solution out of the container and rinse it with MAXIMA Elite solution. Leave the container open to air dry.
- If the lenses have been stored in the container for more than 30 days pour the solution out of the container and repeat all the care procedures before wearing lenses.

Precautions:

- Make sure that the packing is intact before the first use.
- If the crew-top is defective do not use the solution.
- Do not touch the bottle dropper tip since this may contaminate the solution.
- Always close the bottle tight after every use.
- Once the bottle is opened use it for not more than 120 days (4 months).
- Never use the solution which has expired (indicated on the bottle)
- Keep at temperature from 4°C until 25°C (40°F until 77°F)

Warnings:

- Keep out of reach of children
- Never mix MAXIMA Elite solution with other liquids or contact lens care systems.
- If you use contact lenses consult your eye-care practitioner before using any medication.
- Never use tap water with contact lenses.

Concurrent (adverse) reaction:

- If you experience discomfort or irritation remove your lenses, stop using the solution and consult your eye-care practitioner.

Package contents:

- MAXIMA Elite solution is manufactured in 100 and 360 ml bottles including interchangeable lens container.

Ordered By:

Maxima Optics (UK) Ltd.
3 Woodside Lane, London, N12 8RB,
Great Britain

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